

Monthly Maintenance Check Log
(For the Defibtech DDU-100 Semi-Automatic External Defibrillator)

Inspector's Name: _____

AED Location: _____ AED Director: _____

AED Model#: _____ AED Serial #: _____

Date of Inspection: _____ Time: _____ AM PM

1. Check the condition of the unit and accessories.
2. Is the unit dirty or contaminated?

Dirty or Contaminated? Yes No

If "Yes," clean the defibrillator thoroughly, and notify AED Coordinator of possible contamination.

Date Cleaned: _____ Date AED Coordinator notified: _____

3. Check the expiration date on the pads and the battery pack.

Pads: Has the expiration date passed? Yes No

Battery: Has the expiration date passed? Yes No

If "Yes" for either, replace the expired equipment immediately, and notify the AED Coordinator.

Pads Expiration Date: _____ Battery Expiration Date: _____

Date AED Coordinator notified: _____

AED Accessory Kit attached to AED and supplied properly? Yes No N/A

Check the expiration date of extra pads: Current Expiration Date? Yes No N/A

Extra Pads Expiration Date: _____

Remarks, Problems, Corrective Actions: