

Monthly Maintenance Check Log

(For the Cardiac Science Powerheart AED G3 Plus)

Inspector's Name: _____

AED Location: _____ AED Director: _____

AED Model#: _____ AED Serial #: _____

Date of Inspection: _____ Time: _____ AM PM

1. Open the AED lid.
2. Wait for the AED to indicate status: Observe the change of the STATUS INDICATOR to RED. After approximately 5 seconds, verify that the STATUS INDICATOR returns to GREEN.

TURNED GREEN? Yes No

If "No," call the manufacturer for repair. Record date AED was sent for repair and the day it returned.

Date Sent: _____ Date Returned: _____

AED Coordinator notified of date Sent: _____

AED Coordinator notified of date Returned: _____

3. Listen for the voice prompts, and check that the display shows the proper corresponding text prompts.
4. Close the lid and observe the change of the STATUS INDICATOR to RED. After approximately 5 seconds, verify that the STATUS INDICATOR turns to green.

TURNED GREEN? Yes No

If "No," call the manufacturer for repair. Record date AED was sent for repair and the day it returned.

Date Sent: _____ Date Returned: _____

AED Coordinator notified of date Sent: _____

AED Coordinator notified of date Returned: _____

NOTE: On Cardiac Science Powerheart AED G3 Plus, replace battery when the red LED illuminates. When the red LED initially illuminates, upon lid opening or at any time during a rescue, a BATTERY LOW prompt appears. However, the AED is capable of delivering at least 9 more defibrillation shocks after the first BATTERY LOW prompt appears.

AED Accessory Kit attached to AED and supplied properly?	Yes	No
Check the expiration date on the pads: Is the date in the future?	Yes	No
Check the expiration date of extra pads: Is the date in the future?	Yes	No
Expiration Date of Battery in unit?	_____	

Remarks, Problems, Corrective Actions: